MED-COMFORT®

Digital-Fieberthermometer

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Digital Thermometer

Measurement accuracy $\pm 0,1^{\circ}\text{C}$, Waterproof

This digital thermometer measures body temperature quickly and accurately. The measured temperature is easy to read and displays until the thermometer turns off automatically after approximately 10 minutes. A short tone signals the operational readiness and the end of the measurement. The last measured value is automatically stored. The thermometer is waterproof and free of mercury. Not suitable for small children.

The thermometer is produced according to standard EN 12470-3 Clinical thermometers - Part 3.

Measuring range:	32°C to 43°C
Accuracy:	$\pm 0,1^{\circ}\text{C}$
Display :	Liquid crystal display (LCD) LO°C = temperature under 32 °C HI°C = temperature over 43 °C
Battery:	1.5 V button battery (LR-41)
Dimensions:	approx. 129 x 22 x 14 mm (with transparent box)

Manual:

1. Clean with disinfectant wipe before use.
2. Gently press the power button on the thermometer (do not use fingernails).
3. Short signal informs when the thermometer is ready for use.
4. The last measured temperature is displayed, if the button is pressed for 2 seconds.
5. Can be used orally (in the mouth), rectally (in the back) or under the arm.
6. The measurement is terminated when the signal sounds.
7. It is recommended to turn off the unit. The device will automatically turn off after 10 minutes.

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The thermometer is used exclusively for measuring the human body temperature, and should not be left with unattended children (risk of swallowing small parts). Do not bend thermometer, and make sure to protect it from sunlight and heat. Avoid contact with chemicals. Do not use aggressive cleaning agents and disinfectants. Only open to change battery.

Metrological inspection:

Germany: a metrological control has to be carried out according to the Medical Devices Directive (MPVBetrieV) every 2 years. This control can only be carried out by the Manufacturer, a competent authority for metrology or persons, that meet the requirements of MPVBetrieV § 6.

European Community except Germany: Valid for all countries of the European Community, except Germany, are the relevant statutory provisions.

Used batteries must not be disposed in household waste. Consumers are committed to bring the batteries to a suitable collection in trade or municipality. You can also return used batteries to us. Batteries contain potentially pollutants and heavy metals that can harm health and the environment. Batteries are recyclable, they contain important raw materials such as iron, zinc, manganese or nickel.

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Hangzhou Hua'an Medical & Health Instruments Co., Ltd.
Building 2, 1# Fuzhu Nan RD · Wuchang Town
Yuhang District · 310023 Hangzhou, Zhejiang · CHINA

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Shanghai International Holding Corp. GmbH (Europe)