Troponin I Control Kit

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Troponin I control kit

English For human medical use only!  

for quality control procedures of the quantitative in vitro determination with the Troponin I test kit on smart or CUBE laboratory photometer.

Order information	Indication	Kit size
Order number: ST0230	Troponin I test kit	6 tests
Order number: ST2300	Troponin I control kit	1 x 1 ml (decision level)

Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.

Summary

This Troponin I control kit is to be used for internal quality control procedures in examining the accuracy and precision of quantitative Troponin I tests. For in vitro diagnostic use only! The controls are manufactured from human serum. The kit was tested by approved methods and found negative for the HIV I/II Ab, HBsAg and HCV. However, because no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious disease and disposed of accordingly.

Storage and Handling

Unopened vials are stable at 2 - 8 °C until the expiration date on the label. Once opened vials are stable until the expiry date when stored tightly capped at 2 - 8 °C.

Waste Management

Please refer to local legal requirements.

Preparation

Controls are liquid and ready for use. Take care to avoid the formation of bubbles. Use in the same way as a sample. Mix carefully before each use by swirling gently. Pipette enough control material into an empty uncoated Eppendorf or similar cup and pipette out of this cup. Use only disposable pipette tips to avoid contamination.

Warnings and Precautions

For in vitro diagnostic use only. DO NOT INGEST. Avoid contact with skin and eyes. Contains sodium azide, which may react with lead or copper plumbing to form explosive compounds.

Take the necessary precautions for the use of laboratory reagents.

Set the sex to „Control“ before running the QC.

Compare the values indicated in the lot-specific value sheet included in the control kit.

If the result is not within the acceptable range repeat measurement.

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3. Processing of a Troponin I control measurement

- 3.1 Dispense some drops control liquid into serum cup. Aspirate 50 µl from serum cup.
- 3.2 Dispense 50 µl sample INTO LIQUID in ERS cuvette
- 3.3 Apply ERS cap firmly onto ERS cuvette
- 3.4 Place ERS cartridge into laboratory photometer
- 3.5 Start automatic sample processing by pressing the  start button on the smart laboratory photometer, or by closing the door of the CUBE laboratory photometer.

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