# hsCRP Control Kit

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# hsCRP control kit

**English** For human medical use only!  

for quality control procedures of the quantitative in vitro determination with the hsCRP test kit on smart or CUBE laboratory photometer.

Order information	Indication	Kit size
Order number: ST0105	hsCRP test kit	32 tests
Order number: ST1005	hsCRP control kit	1 x 1 ml (decision level)

**Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

## Summary

This hsCRP control kit is intended for use in the clinical laboratory as a control for quantitative determination of hsCRP. It is prepared from human serum and was found to be nonreactive when tested with approved methods for HbsAg, anti HIV1/2 and anti HCV. However, this material should be handled as though capable of transmitting infectious disease.

## Storage and Handling

Unopened vials are stable at 2 - 8 °C until the expiration date on the label. Thoroughly mix the content of the vial before each use by gently inverting it. After use please cap vial immediately and put back into refrigerator. Once opened vials are stable until the expiry date when stored tightly capped at 2 - 8 °C.

We recommend not to pipette directly from the original bottle as well as using single-use pipetting tips to avoid contamination. Dispense about 5 drops into a serum cup and pipette out of it. Dispose cup after use.

## Waste Management

Please refer to local legal requirements.

## Preparation

The controls are liquid and ready to use. Thoroughly mix the contents of the vial before each use by gently inverting for several minutes. For further details on how to use the control kit please refer to the description inside this package insert.

## Control Values

The values are traceable to the reference material ERM DA474/IFCC.

## Warnings and Precautions

Set the sex to „Control“ and the samplotype to „Serum“ before running the QC.

Compare the results with the values indicated in the lot-specific value table.

If the result is not within the acceptable range repeat measurement. If result is again not within the acceptable range please contact your distributor.

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#### 3. Processing hsCRP control measurement with pipette

- 3.1 Dispense some drops control liquid into serum cup. Aspirate 20 µl from serum cup.
- 3.2 Dispense 20 µl sample INTO LIQUID in ERS cuvette
- 3.3 Apply ERS cap firmly onto ERS cuvette
- 3.4 Place ERS cartridge into laboratory photometer
- 3.5 Start automatic sample processing by pressing the  start button on the smart laboratory photometer, or by closing the door of the CUBE laboratory photometer.

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