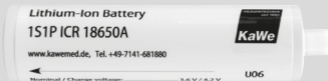
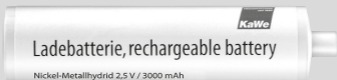
de	Gebrauchsanweisung	Ladebatterie
en	User's manual	Rechargeable batteries
fr	Mode d'emploi	Piles rechargeables
it	Istruzioni per l'uso	Batterie ricaricabili
es	Instrucciones de empleo	Pilas recargables
pt	Manual de operação	Pilhas recarregáveis
ru	Руководство по применению	Аккумулятор

de

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Technical data:

REF	12.80110.712***	12.80110.722***	12.80120.712
Voltage [V]	2.5	2.5	3.5
Capacity [mAh]	1600	3000	750
Charging time [h]	approx. 8*	approx. 14*	approx. 4*

REF	12.80220.722	12.80120.742
Voltage [V]	4.2	3.5
Capacity [mAh]	2200	700
Charging time [h]	approx. 9*	approx. 12**

EN

- * Charging time of fully discharged rechargeable battery using either the KaWe MedCharge® 3000 (not Li-Ion) or 4000 charging stations.
- ** Charging time of fully discharged rechargeable battery using an electric wall socket.
- *** These rechargeable batteries MAY NOT be used with the 2.5 V LED laryngoscope.

Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product meets EC Standards 93/42/EWG (standards for medical products).

Please read this user's manual thoroughly and carefully before using this product and heed the given care instructions. Familiarize yourself fully with the proper operation of this device before attempting to use it. Save this user's manual for future reference and pass it on to the next user of this device. For questions the connection or operation of this device, please contact customer service.

This device may only be used properly as described here. Noncompliance with the generally applicable regulations and these instructions releases the manufacturer from liability for any resulting damages.

Intended use:

These rechargeable batteries serve as the energy source for the intended KaWe products. The REF 12.80110.712 and 12.80110.722 rechargeable batteries may not be used for the 2.5 V LED laryngoscope.

Safety information:

Improper use or use of damaged batteries may result in the leakage of fumes. Ventilate the area and seek medical attention for health complaints. These fumes can irritate the airways.

Please heed the following instructions:

- Charge the batteries only with the unit intended for this purpose.
- Not allowed: sterilization, autoclave treatment, gas sterilization and Sterrad
- Do not expose to direct sunlight
- Keep away from children
- Store in a cool and dry place
- Never immerse the batteries in liquids.
- After successfully charging the batteries, unplug the charging unit from the wall socket.
- Do not cover the batteries or charging units (with carpet, pillows, bedding etc.)

EN

Risk of fire! Risk of explosion!

Never use damaged, defective or deformed batteries. Do not open, damage or drop the batteries. Do not charge the batteries near acids or easily-flammable materials. Protect the batteries from heat and fire. Only use the batteries at an ambient temperature of between 5°C and +40°C. Never lay the batteries on heaters or expose them to direct sunlight for an extended period. After heavy use, first allow them to cool. Short circuit – Do not bridge the battery electrodes with any kind of metal. The batteries are to be wrapped up or packaged (in a plastic bag or box) for disposal, transport or storage. Alternatively, the battery electrodes may also be covered with tape.

Operating instructions:

The batteries come only partially charged and must be fully charged before the first use. Only use original KaWe handles.

NiMH-rechargeable battery

With NiMH technology, a lazy battery effect can occur similar to the classic memory effect of NiCd technology. The cause of this is the residual energy present in the battery when the battery is recharged. This effect can be almost completely removed by completely discharging the battery and then recharging it from time to time. If the battery no longer provides enough energy to power your device, turn off the device immediately and charge the battery. (Avoid a deep discharge of the rechargeable battery, as this will destroy it.)

EN

Please note that these batteries will self-discharge. If they are not needed for a longer period of time (used and charged) they will self-discharge, lose capacity and can sustain permanent damage. The manufacturer is not liable for such damage. The rechargeable batteries should therefore be stored with a charge of no less than 50% if they are to be in storage for longer than one month.

Always heed the given charging time, as overcharging will damage the battery. (KaWe MedCharge® 3000 and 4000 charging stations are equipped with an integrated automatic shut-off function.)

NiMH rechargeable batteries lose capacity at low temperatures (less than 0°C).

When not in use, charge the battery every 3 months.

Li-ion battery

The Li-ion battery can be recharged at any time without shortening the battery life.

Interrupting the charging process does not harm the battery.

A drastically shortened battery life after a full charge is a sign that the battery is depleted in need of replacement. Only use original replacement battery packs.

When not in use, charge the battery every 3 months.

Disposal:

Disposal/recycling instructions are printed on the device itself as well as on the packaging.

Electric or electronic devices that are damaged or require disposal must be brought to a proper recycling centre.

Environmentally friendly disposal of batteries:

Batteries must be disposed of properly. Bring defective batteries to a dealer for disposal. Batteries must be removed from devices before they are disposed of. Damaged batteries can be harmful to the environment and your health if they leak poisonous vapors or liquids. Therefore, never mail or otherwise send out a damaged battery. Please consult your local disposal facility/recycling centre. Ensure that the batteries are completely discharged before you dispose of them. We recommend covering the battery contacts with tape in order avoid a short circuit. Do not open the battery.

EN

Symbol key:

	Manufacturer		Caution!
	Date of manufacture		CE conformity label
	Batch code		Do not dispose of batteries in household waste.
	Heed the user's manual		Separate disposal of electric and electronic devices
	Separate disposal of electric and electronic devices		


Contact information:

Address or tel. no. of your dealer or dial +49 7141 68188-0

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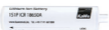REF	12.80110.712***	12.80110.722***	12.80120.712
Volt [V]	2,5	2,5	3,5
Capacité [mAh]	1600	3000	750
Durée de charge [h]	env. 8*	env. 14*	env. 4*

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KIRCHNER & WILHELM GmbH + Co. KG