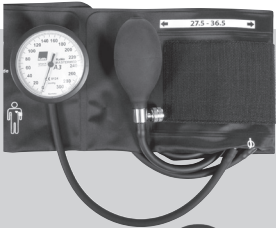
- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de uso
- pt** Manual de operação
- ru** Руководство по применению

**MASTERMED**  
(A1/A1+, A2/A2+, A3, C)

**CE**  
1304

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Telefon: 0800 - 10 10 871 seit 1921

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## Content

<b>1. Explanation of the symbols</b> .....	13
<b>2. Application</b> .....	13
<b>3. Intended use</b> .....	13
<b>4. Intended patient group</b> .....	13
<b>5. Contraindications</b> .....	13
<b>6.  Warnings and precautions</b> .....	13
<b>7. MASTERMED C Assembly</b> .....	15
7.1. Wall installation.....	15
7.2. Stand installation.....	16
<b>8. Auscultatory blood pressure measuring</b> .....	17
8.1. Basic rules for the patient support.....	17
8.2. Preparation.....	17
8.3. Attaching the arm cuff.....	17
8.4. Performing the measuring.....	17
<b>9. Cleaning &amp; Disinfection</b> .....	18
9.1. Blood pressure measuring devices.....	18
9.2. Arm cuffs.....	18
<b>10. Transportation and Storage</b> .....	18
<b>11. Technical Data</b> .....	18
<b>12. Accessories</b> .....	18
12.1. Accessories A1 / A1+.....	18
12.2. Accessories A2 / A2+ and C.....	18
12.3. Accessories A3.....	19
<b>13. Maintenance</b> .....	19
<b>14. Disposal</b> .....	19
<b>15. Warranty</b> .....	19
<b>16. Contact</b> .....	19
<b>17. Basic UDI-DI</b> .....	19

Dear Customer, thank you for choosing our innovative aneroid blood pressure measuring device. Our products are known for their high quality and longevity.

This KaWe product meets the requirements of the Standard ISO 81060-1 and the Regulation (EU) 2017/745 (European Medical Devices Regulation) and belongs to the medical device class Im according to this Regulation.

**Please read this User's Manual thoroughly and completely before use and follow the given application and care instructions.**

## 1. Explanation of the symbols

	Caution!
	Follow the User's Manual
	Temperature limits
	Air humidity limit
	For reuse on a single patient
	No latex contained
	Manufacturer & production date, if necessary
	Production date
	Item number
	Batch description
	Serial No.
	Medical device
	UDI data carrier
	CE marking of conformity
	Natural rubber latex contain

## 2. Application

This blood pressure measuring device and its accessories are only to be used by physicians and trained professionals for monitoring blood pressure.

## 3. Intended use

This device is intended for the non-invasive measurement of systolic and diastolic blood pressure in humans.

## 4. Intended patient group

People with arm circumference of min 13.8 cm (infants) to max 46 cm (large adults).

## 5. Contraindications

Any use other than described in the intended use shall be deemed improper. The manufacturer shall not be liable for any resulting damages. The user alone bears the risk.

## 6. Warnings and precautions

- When the cuff is not connected to the gauge, always use a tube stopper to prevent accidental penetration of liquid. The penetration of liquid into the tube may cause damage to the gauge.
- The device must not be used in the area of strong magnetic fields (MRI).
- Periodically inspect and replace damaged or used cuffs.
- Use only the appropriate size cuffs. The arterial pointer must be in the marked area, otherwise the measurement readings may be incorrect.
- Single-patient cuffs are intended for use **only on one single patient**.
- Do **not** immerse the device **in liquid** to clean it.
- Do **not sterilize** the device.
- The blood pressure measuring device and the cuff must be handled carefully and must not be exposed to sharp objects (scissors, needles etc.) because they can damage the device and the cuff.

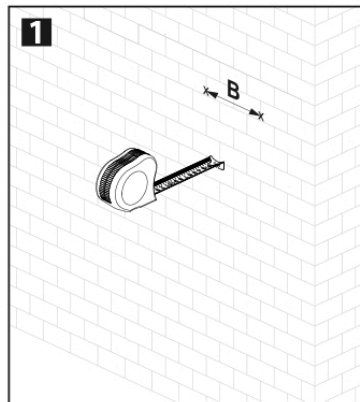
- Protect the device from strong shocks (do not bump or drop it).
- The pointer on the gauge must point within the tolerance range of the zero position when there is no air in the system.
- Always let all of the pressure out of the cuff before storing the device.
- **Never** inflate to more than 300 mmHg!
- Measurement time should be **maximum 2 minutes**.
- The **break** between two measurements should be minimum **2 minutes**.
- When replacing or repairing parts, please ensure that only original KaWe replacement parts are used.
- Make sure that the tube system is airtight.


Modification of the device and/or accessories is **not permitted!**  
Otherwise that can lead to measuring errors.

## 7. MASTERMED C Assembly


### 7.1. Wall installation

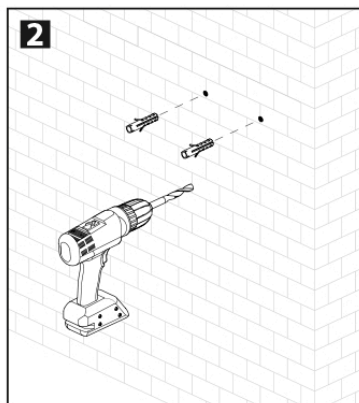1) Mark the position of the drill holes for the wall mount.

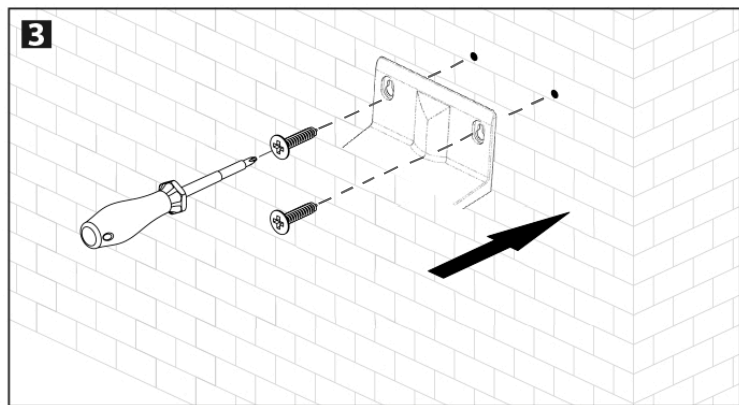
2) Drill the holes ( $\varnothing$  6 mm) at a distance  $B = 110$  mm.


3) Screw on the wall mount.

There is space behind the manometer to store the cuff.


## 7.2. Stand installation

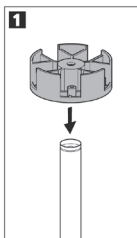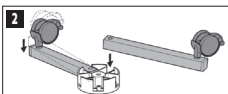
1) Put on the black plastic cap.

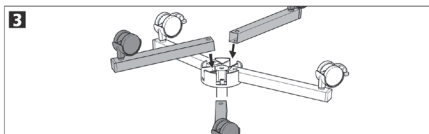
2) Mount two foot tubes **with brake castors** opposite each other.

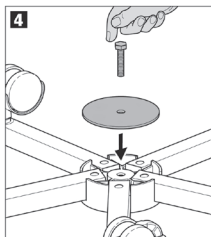
3) Mount three foot tubes **without brake castors**.


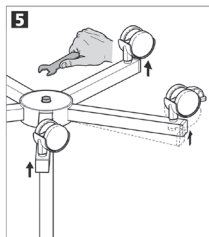
4) Screw in hexagon screw.

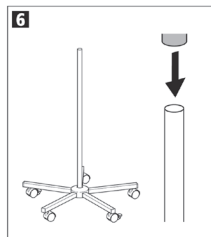
5) Tighten hexagon screw.

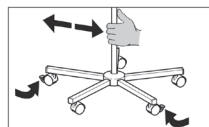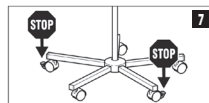6) Place the manometer onto the stand post.

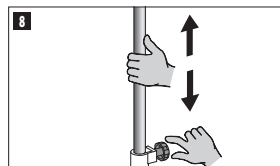7) For a firm stand, lock the brake castors. To move forward, release the brake castors.

8) Adjust the height as required.

## 8. Auscultatory blood pressure measuring

### 8.1. Basic rules for the patient support

The patient should:

- rest for 5 minutes before the first measurement
- sit comfortably
- do not cross their legs
- lean or otherwise support the back and arms
- be as relaxed as possible during the blood pressure measurement and should not talk

### 8.2. Preparation

- 1) It is recommended to use the following phases of Korotkoff sounds for auscultatory measurement:
  - in adults, phase V (K5)
  - in children aged 3 to 12 years, phase IV (K4)
  - in pregnant women, phase V (K5), unless noises are heard with the cuff deflated, in such cases phase IV (K4) is used

**K5** is the point at which the sounds heard with a stethoscope are no longer audible

**K4** is the point at which the sounds heard with a stethoscope vary from clear tapping to muffled.

- 2) Before taking a measurement, insure that the blood pressure measuring device and the cuff are in **perfect working order**.
- 3) The cuff must be chosen to fit the arm circumference:

Arm cuff	Arm circumference
Infants	13.8 - 21.5 cm
Children	20.5 - 28.5 cm
Adults	27.5 - 36.5 cm
Large adults	35.5 - 46.0 cm

- 4) A stethoscope is required separately for auscultatory measurement.
- 5) Help the patient to relax and remind them not to speak or move during the measurement.

### 8.3. Attaching the arm cuff

- 1) The cuff must not be applied over wounds as this may cause further injury.
- 2) Blood pressure must always be measured on the arm with the higher pressure values. For this purpose, measure the blood pressure on both arms first and then always on the arm whose blood pressure value is higher.
- 3) Pull the cuff up over the upper arm. Thereby the tube must be pointing downwards. Make sure that the center of the cuff on the upper arm is at the same level as the right atrium of the heart.
- 4) The artery sign  $\Phi$  on the cuff (KaWe logo visible from the outside) must be on the inside of the upper arm in the area of the artery.
- 5) Tighten the cuff firmly but not too tightly (leave 1-2 cm free) and lock it with the Velcro fastener.

### 8.4. Performing the measuring

- 1) Close the release valve on the pressure ball by turning the screw to the right.  
But do **not overtighten** the screw.
- 2) Pump up the cuff with the pressure ball quickly and rhythmically until it exceeds the systolic pressure.  
But **never** inflate to more than 300 mmHg!
- 3) Open the release valve slowly so that the pressure drop at the pressure gauge is approx. 2-3 mmHg/s.  
Read the systolic and diastolic pressure **perpendicular to the scale** to avoid reading error.
- 4) After the measurement is finished, open the release valve fully to let the air escape completely from the cuff.
- 5) Remove the cuff from the arm.
- 6) Stow the gauge and/or the cuff:
  - MASTERMED A series: in the bag supplied
  - MASTERMED C series: in the holder

## 9. Cleaning & Disinfections

### 9.1. Blood pressure measuring devices

#### **Cleaning:**

To clean the device, use only a soft, dry cloth.

The ball, valve and tubes are to be wiped clean with a mild detergent and a damp cloth.

#### **Disinfection:**

For disinfection of the device by wiping (minimum exposure time of at least 5 minutes) it is recommended to use the disinfectant Mikrozyd® sensitive liquid (by Schülke & Mayr).

### 9.2. Arm cuffs

#### **Cleaning:**

If necessary, wipe the cuff clean with a damp cloth. Mild soapy water may be used for cleaning.

#### **Disinfection:**

The reusable cuffs can be disinfected with medical alcohol 70%. After disinfection, let the cuffs air dry.

## 10. Transportation and Storage

Transport and store the devices incl. accessories in a way so that they are protected from dust, moisture, and contamination.

Storage conditions Blood pressure measuring devices	-30 °C up to +70 °C, 15 - 85 % rel. humidity
Storage conditions Arm cuffs	-5 °C up to +40 °C, 15 - 85 % rel. humidity

## 11. Technical Data

Measurement range	0 – 300 mmHG
Measurement accuracy	± 3 mmHG or 2 % of the reading value (the higher value applies)
Operating conditions Blood pressure measuring devices	+10 °C up to +40 °C, 15 - 85 % rel.
Operating conditions Arm cuffs	humidity
Lifespan Blood pressure measuring devices	A series: 10 years C series: 5 years
Marking "shock protected"	Proper functionality is tested up to a drop height of 1 m

## 12. Accessories:

For arm circumference of each cuff size, see section 8.2 on page 17.

### 12.1. Accessories A1 / A1+

#### **reusable 1 tube arm cuffs**

04.85104.211 - Infants

04.85104.221 - Children

04.85104.231 - Adults

04.85104.241 - Large adults

#### **Single patient 1 tube arm cuffs**

04.85140.221 - Children

04.85140.231 - Adults

04.85140.241 - Large adults

### 12.2. Accessories A2 / A2+ and C

#### **reusable 2 tube arm cuffs**

04.85204.211 - Infants

04.85204.221 - Children

04.85204.231 - Adults

04.85204.241 - Large adults

## 12.3. Accessories A3

### reusable 2 tube arm cuffs

04.86600.221 - Children

04.86600.231 - Adults

04.86600.241 - Large adults

## 13. Maintenance

A **metrological check** should be carried out at least every **2 years**. This can be done either by the manufacturer or by authorized service providers. Please take note of your national requirements regarding this.

For users in **Germany**, metrological check is **mandatory** in accordance with the MPBetreibV.

## 14. Disposal

The product is to be disposed of in accordance with the local requirements.

## 15. Warranty

If used properly and in accordance with our user's manual, the warranty period is two years from the date of purchase. For further questions or possible repairs, please contact your specialist dealer.

## 16. Contact

Contact your specialist dealer or the manufacturer (see reverse for address).

Further information: [www.kawemed.com](http://www.kawemed.com)

## 17. Basic UDI-DI

4030155KaWe04SQ

### Note on users & patients

All serious incidents related to the device shall be reported immediately to the manufacturer and to the competent authority of the EU member state in which the user and/or the patient is established.

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**KIRCHNER & WILHELM GmbH + Co. KG**